



## VT Select Bag-Valve-Mask (BVM) resuscitator



Anaesthesia • Resuscitation and Emergency Care

## VT Select BVM resuscitator

As demonstrated by previous studies<sup>1</sup>, manual resuscitation can be highly variable from provider to provider. The VT Select helps to minimise the variability in treatment by using a timed refill valve which can be selected at the back of the bag.

It is recommended by the American Heart Association<sup>2</sup> to deliver 10 breaths per minute (BPM) of 500–600 ml in adult resuscitation. When Activated, the valve restricts the refill of the bag to approximately 4 seconds, providing approximately 10 BPM to the patient, helping to reduce the risk of hyperventilation.

When the valve is activated, a one handed squeeze of the bag will deliver approximately 500–600 ml to the patient. Alternatively, the valve can be switched to Standard, removing the delayed refill and the BVM can be used as normal.

The VT Select is smaller than most adult bags at only 1200 ml, which also helps to reduce the risk of hyperinflation or barotrauma.

### Specifications

Body mass range	> 40 kg
Delivered volume range	(1 hand) 530 ml / (2 hand) 690 ml

### Optional accessories

- Adjustable PEEP valve
- Dial manometer
- BVM mask strap

### Clear anatomical cushion mask

For an effective seal and maximises patient visibility



BVM resuscitators are single-use and latex free.



### Smaller volume bag

The 1200 ml bag has raised finger ridges and integrated low profile strap

### Reservoir bag and oxygen tubing

### Valve

#### Activated position

The valve is restricted to produce a controlled tidal volume and timed refill

#### Standard position

The valve has no restriction and is operating as a standard BVM

#### SELECT VALVE POSITIONS:



STANDARD



ACTIVATED

Note: Not currently available for CE regions.

Code	Description	Box. Qty
7159094	VT Select BVM resuscitator, adult, 1.2 L bag, size 5 mask	8
7159093	VT Select BVM resuscitator, adult, 1.2 L bag, dial manometer, size 5 mask	8
7159092	VT Select BVM resuscitator, adult, 1.2 L bag, adjustable PEEP valve, size 5 mask	8
7159091	VT Select BVM resuscitator, adult, 1.2 L bag, adjustable PEEP valve, dial manometer, size 5 mask	8
7162111	Dial pressure manometer	20
7162112	Angled pressure manometer	20
7163000	BVM mask strap	10

#### Reference:

1. American Medical Association, Benjamin S. Abella, MD, MPhil Et Al, 2005. Western Journal of Emergency Medicine, Benjamin Dafilou, BA Et Al, 2020.
2. American Heart Association, 2020. www.ahajournals.org/doi/epub/10.1161/CIR.0000000000000916

VT Select is manufactured by Pulmodyne Inc and distributed by Intersurgical Ltd.

Please think before you print. Save energy and paper. If you must print this information sheet please print it double sided.

IS5.41 • Issue 3 03.25



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK  
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com

UK • Australia • Benelux • Brazil • Canada • China • Colombia • Czech Republic • Denmark • France • Germany • India • Ireland • Italy  
Japan • Lithuania • Malaysia • Mexico • Norway • Philippines • Portugal • Russia • South Africa • Spain • Sweden • Taiwan • Turkey • USA