




## 7000 Series Respiratory Humidifier



**Critical Care** ▪ Respiratory Humidifier

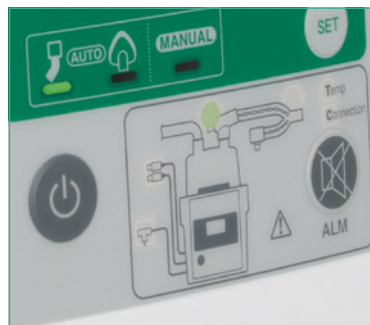

## The 7000 Series Respiratory Humidifier

The 7000 Series Respiratory Humidifier, will provide a further customer option for active humidification and a perfect platform for our extensive range of heated wire products and accessories.

The 7000 Series Respiratory Humidifier offers a number of key benefits:

- Simple intuitive device whilst providing advanced options
- The user has the choice of which temperature is displayed as default option
- Includes a Manual mode option, allowing users the ability to pre-set airway and chamber temperature
- Mode default - reverts to last setting used
- An alarm indicator panel with icon display of alarm location and priority
- A dual temperature probe is included in the package

🎥 Humidifier set up video available at [www.intersurgical.com](http://www.intersurgical.com)



### Locking button

One touch mounting allows for easy insertion and removal of the humidification chamber

### Advanced heater plate

Prevents the heater plate warming if a chamber is not present

### Mode selection


IPPV, NIV and a Manual mode which allows the user to set the desired temperatures

### Configurable display

Choice of chamber or patient end airway temperature

### Alarm indicator panel

Highlights alarm location and priority

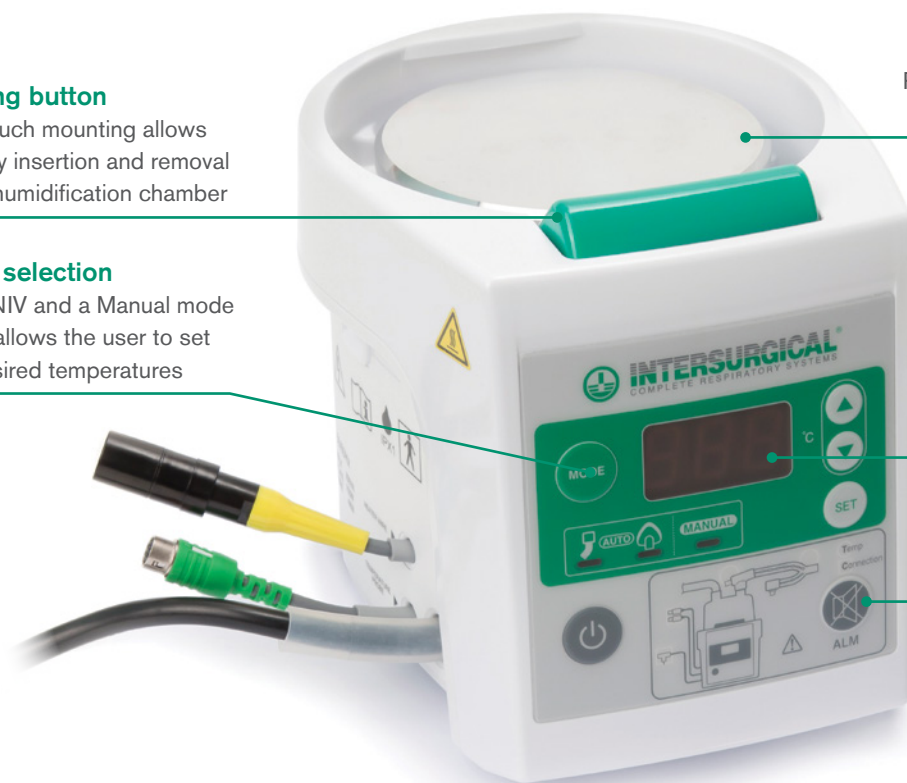

| Code    | Description   | Box Qty. |
|---------|---|----------|
| 7000000 | Respiratory humidifier (UK) including dual temperature probe                                      | 1        |
| 7000001 | Respiratory humidifier (Japan) including dual temperature probe                                   | 1        |
| 7000002 | Respiratory humidifier (EU1) Western Europe including dual temperature probe                      | 1        |
| 7000003 | Respiratory humidifier (EU2) Eastern Europe, Turkey and Portugal including dual temperature probe | 1        |
| 7700000 | 7000 Series Respiratory Humidifier dual temperature probe assembly                                | 1        |

Make an enquiry



## Chambers for heated humidification

We offer a comprehensive range of active humidification chambers for use with the more commonly found humidifiers used in the intensive care areas, including the Intersurgical 7000 Series Respiratory Humidifier.

Compatible with all our heated wire products, all three chamber options feature 22mm male connectors and a clear polycarbonate body for instant and accurate visible fluid level assessment.

- **The dual float auto fill** humidification chamber can be used in a variety of respiratory support therapies ranging from CPAP through to full mechanical ventilation including high frequency oscillation for all size patients. This chamber offers the clinician a constant water level, as the used fluid is automatically replaced from a reservoir bag. As a result the system compressible volume does not fluctuate, making the auto fill chamber the first choice in all intensive care areas from neonatal through to adult.
- **The manual fill** chamber feature a clamp along the fill line which isolates the chamber from the fluid reservoir bag. This chamber requires the clinician to monitor the water level and open the clamp when more fluid is required. Seen as a lower cost alternative and is sometimes used in high driving pressure applications (eg. nFlow™).
- **The low volume** manual fill chamber also features the fill line clamp. With very low compressible volume, it is ideally suited for applications like advanced high frequency ventilation for all sizes of patient.



2310000



2320000



2330000

| Code         | Description   | Box Qty. |
|--------------|---|----------|
| 2310000 (S*) | Dual float auto fill humidification chamber with fill set             | 30 (30*) |
| 2320000      | Manual fill humidification chamber with fill set and clamp            | 30       |
| 2330000      | Low volume manual fill humidification chamber with fill set and clamp | 30       |

[Make an enquiry](#)

## Humidifier electrical adaptor leads

A range of leads for electrical connection of the heated wire breathing system to the humidifier.



5600000



5601000



5603000



5604000

| Code    | Description  | Humidifier          | Box Qty. |
|---------|--|---------------------|----------|
| 5600000 | Electrical adaptor lead for single heated wire breathing systems with black plug | MR850™, 7000 Series | 1        |
| 5601000 | Electrical adaptor lead for dual heated wire breathing systems with black plug   | MR850™, 7000 Series | 1        |
| 5603000 | Electrical adaptor lead for single heated wire breathing systems with grey plug  | MR730™              | 1        |
| 5604000 | Electrical adaptor lead for dual heated wire breathing systems with grey plug    | MR730™              | 1        |

[Make an enquiry](#)

**S** Sterile option available


(S\*) Add an S to the seven digit code number for the sterile version of this product eg. 5000000S (sterile box quantity is shown in brackets).

MR850, MR730, MR850, MR500 and MR600 are trademarks of Fisher & Paykel Healthcare.



## Flextube™ breathing systems for active humidification with heated wire


Breathing systems are available with single or dual heated wires and can be supplied complete with or without an auto fill humidification chamber. All Intersurgical active humidification breathing systems are suitable for use with the majority of humidification base units found in intensive care areas.

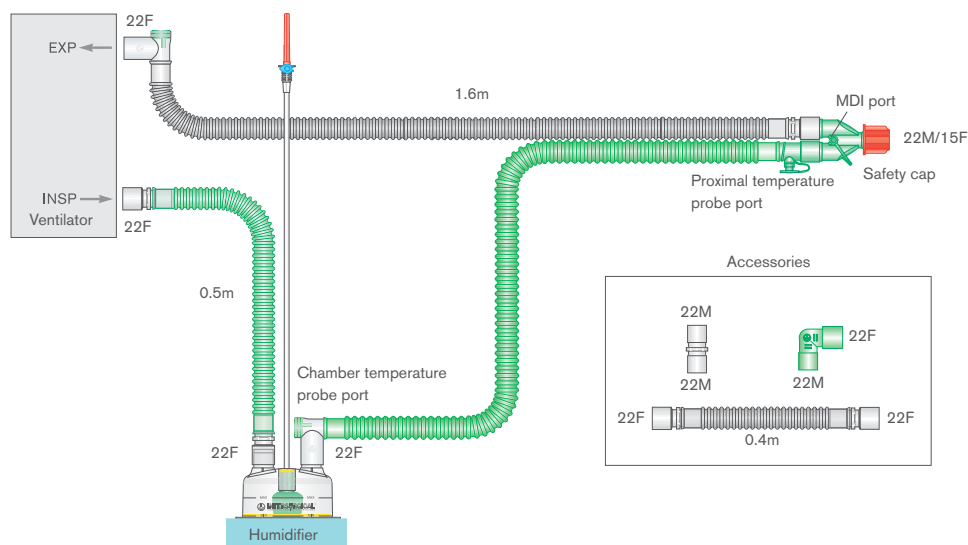

### Dual heated wire

👁 system illustrated

| Code           | Description   | Box Qty. |
|----------------|---|----------|
| 2026000 (S*)   | 22mm Flextube dual heated wire breathing system with limbs. $\geq 1.6\text{m}$                                    | 10 (12*) |
| 👁 2026310 (S*) | 22mm Flextube dual heated wire breathing system with auto-fill chamber and 0.4m and 0.5m limb. $\geq 1.6\text{m}$ | 7 (6*)   |

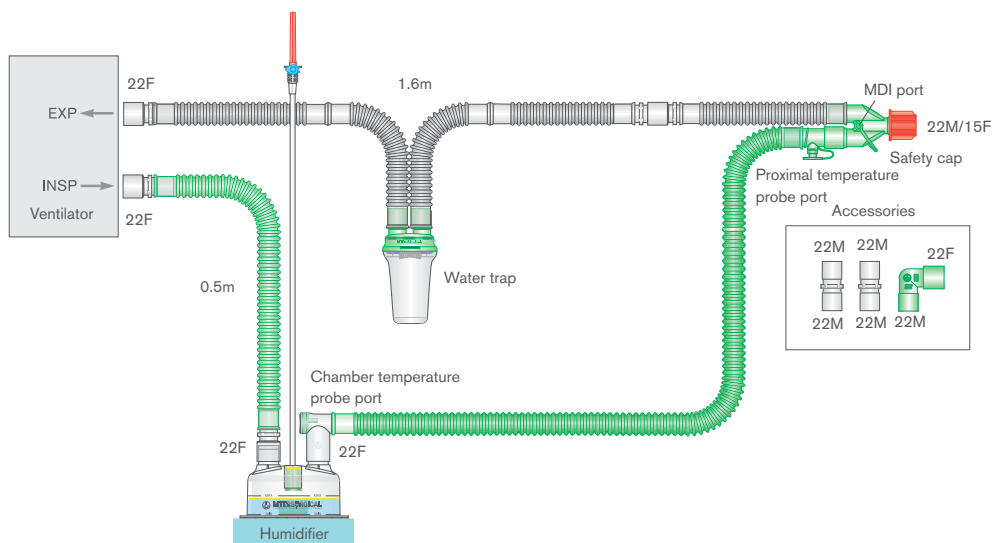

### Single heated wire

👁 system illustrated

| Code           | Description  | Box Qty. |
|----------------|--|----------|
| 2025000        | 22mm Flextube single heated wire breathing system with water trap and limb. $\geq 1.6\text{m}$                         | 10       |
| 👁 2025310 (S*) | 22mm Flextube single heated wire breathing system with water trap, auto-fill chamber and 0.5m limb. $\geq 1.6\text{m}$ | 7 (5*)   |

We have a wide choice of breathing systems for active humidification for adults, paediatrics and neonatal patients. To see the range contact us through our sales team or refer to the website [www.intersurgical.com](http://www.intersurgical.com)



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK

T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 [info@intersurgical.com](mailto:info@intersurgical.com) [www.intersurgical.com](http://www.intersurgical.com)



The manufacturer Intersurgical Ltd is certified to ISO 9001:2015, ISO 13485:2016 and ISO 14001:2015

**Please think before you print**  
Save energy and paper.  
If you must print this information sheet please print it double sided.

IS1.29 • Issue 6 07.19